


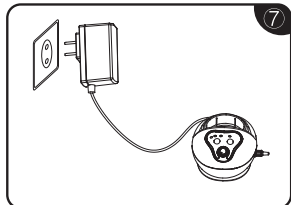
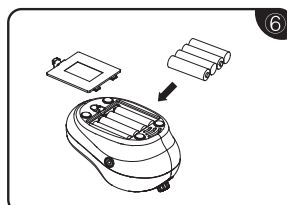
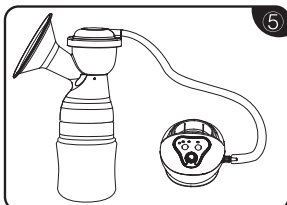
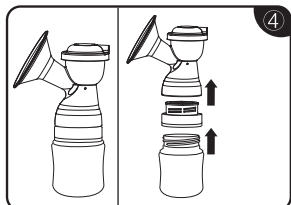
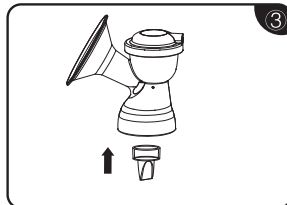
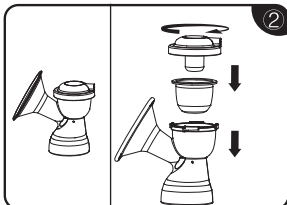
**Fig./Rys./Рис. 2**

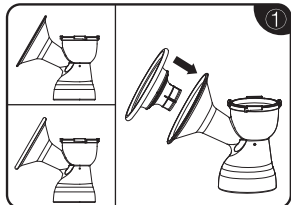**Fig./Rys./Рис. 3**

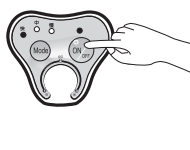**Fig./Rys./Рис. 4**

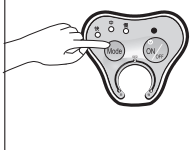**Fig./Rys./Рис. 5**

## Instruction Manual

### Electric Breast Pump EasyStart

#### Model 12/215

This electric breast pump is designed for expressing excess milk. It is not intended for expressing milk when a hungry baby cannot empty the breast themselves, i.e. in all cases when your milk ducts are blocked. Doctors recommend breast milk as the best food for babies in the first year of life, which should be combined with solid food from 4-6 months of age. Remember that your milk is the food that fully meets your baby's needs and it has antibodies that protect your baby from infections and allergies.

#### WHAT IS IN THE SET, fig 1

- |                                 |  |
|---------------------------------|--|
| 1. Upper diaphragm case         | 7. Adapter for wide- and narrow-neck bottles |
| 2. Diaphragm – 1 pc             | 8. Milk container 120 ml                     |
| 3. Silicone massage cushion     | 9. Tube                                      |
| 4. Funnel cover                 | 10. Electric unit                            |
| 5. Breast pump body with funnel | 11. Connector                                |
| 6. Silicone valve – 1 pc        | 12. Power adapter                            |

#### HOW TO ASSEMBLE / DISASSEMBLE / CHANGE BATTERY Fig 2

- 2.1. Place the silicone massage cushion on the funnel. Make sure it fits tightly to the edges of the cup.
- 2.2. Place the diaphragm in the body with the funnel and close the upper diaphragm case.
- 2.3. Place the silicone valve, and make sure it fits tightly to the edges of the hole.
- 2.4. Screw the milk container onto the breast pump body using the adapter.
- 2.5. Attach one end of the silicone tube to the breast pump body and the other to the connector on the electric unit.
- 2.6. The device can be powered by four AA alkaline batteries (batteries are not included in the set).
- 2.7. While at home, you can use the breast pump powering it via a power adapter.

#### HOW TO USE BREAST PUMP, figs 3, 4, 5

Make sure that the breast pump has been cleaned and disinfected before. Wash your hands and breast thoroughly. To check if the milk ducts aren't clogged, express a little bit of milk by squeezing the nipple gently.

Connect the breast pump to the power supply. Press the on/off button (fig.3). If the breast pump is turned on, the indicator above the button shall light. To turn the breast pump off, press the on/off button again. After turning it off, the indicator will go out.

Place the breast pump against the breast so that the silicone massage cushion fits the breast evenly and prevents airflow. If there is an air leak, the effectiveness of the breast pump can be significantly reduced. Always hold the bottle upright so that the breast milk can flow to the bottom of the bottle freely.

The breast pump allows you to adjust the rhythm and strength of the suction to suit your individual needs.

Press the mode button (fig. 4) to regulate the suction rhythm and select one of the three modes:

• **express** – start-up mode

In the start-up mode, the breast pump mimics rapid and shallow sucking movements of the baby occurring during the stimulation phase. The start-up mode is used to stimulate lactation. During the stimulation phase milk starts to flow out from the breast.

• **natural** – natural mode

The natural mode imitates slower, deeper and natural suckling movements that occur during effective suckling of the baby.

• **strong** – strong mode

The strong mode imitates slower, very deep suckling movements occurring during your baby's effective suckling.

When the milk starts to flow, the baby must put more effort into suckling. Milk slowly becomes denser and more filling; that is why the suckling rhythm and the pace slow down.

Thanks to the knob (fig. 5) you can adjust the suction power smoothly.

1. low suction strength
2. medium suction strength
3. high suction strength

Express milk from one breast until you feel that milk starts to flow slower. Place the breast pump against the other breast and continue expression. After finishing, unscrew the bottle from the breast pump and attach the teat with the cap. The milk is ready for your baby.

#### HOW TO CLEAN AND DISINFECT BREAST PUMP PARTS

##### Cleaning process:

Disassemble the following parts of the breast pump: upper diaphragm case (1), diaphragm (2), breast pump body with funnel (5), silicone massage cushion (3), funnel cover (4), silicone valve (6), container (8) and adapter (7) before using the breast pump for the first time. Wash in hot water with mild detergent and rinse thoroughly. Disinfect by placing it in boiling water for about 5 minutes. Dry thoroughly before assembly and subsequent use. **NOTE!** After disinfection wait until all parts have cooled down. Do not start assembling the body when the parts are warm or moist as it can cause breast pump malfunction. Check that the individual parts of the breast pump are correctly attached.

**Cleaning instructions:** Disconnect the silicone tube and the connector from the breast pump. These elements do not require cleaning because the diaphragm protects them from contact with food. All parts of the breast pump should be washed by hand in hot water with mild detergent. Do not use strong detergents or rough materials that can scratch the surface. Rinse with clean, hot water.

**NOTE!** Always remove the anti-return valve by grasping its round base with two fingers. Never grasp the tip of the valve, as it can be damaged what can weaken the suction power. Wash the non-return valve very gently and do not allow food to dry inside it.

**NOTE!** Do not wash the silicone tube and the connector and especially do not wash electrical components. Wipe with a damp cloth.

#### HOW TO CLEAN AND DISINFECT bottle/container

Thoroughly wash and disinfect the milk container and its components before the first and each subsequent use. Place it in boiling water for 5 minutes to ensure hygiene. Clean before each use. Boiling and hard water may create a limescale effect. Wash all parts thoroughly with clean water, including areas that may not be visible but through which food has passed, to remove any food residue. After use, clean the milk container of milk residues, rinse, wash and then disinfect in a steam/electric steriliser or boiling water. The milk container can be used in a dishwasher (max. temp. 60°C). Do not use antibacterial agents for cleaning. Take special care when heating food in a microwave. Always stir heated food to ensure even distribution of temperature and check the temperature before feeding. If you heat food in a microwave, remember that the milk container must be open. The product should not be placed in a preheated oven or heated on a gas or electric stove.

**For your child's safety and health. Note!** Check the product before each use. Throw away at the first signs of damage or weakness. Warning! This product is not a toy. Keep out of reach of children. ALWAYS check food temperature before feeding. Maintain maximum hygiene when using containers. DO NOT boil the breast milk. DO NOT add fresh breast milk to defrosted milk. NEVER refreeze previously defrosted milk. NEVER fill containers with hot liquids. ALWAYS use this product under adult supervision. For breast milk only. Do not use it to prepare modified milk.

##### WARNING!!!

The device is intended for personal use only. Use by more than one person may pose a health risk. The device comes with a 24-month manufacturer's warranty. Please keep your proof of purchase of the device. Before using, read the user manual and follow the rules of safe use. Keep this user manual together with the device. The expected lifetime of the product is five years. The expected lifetime of silicone components is no more than two years. Silicone components may change their colour permanently. Replace silicone components after finding discolouration or after the estimated time of use.

FOLLOW THE INSTRUCTIONS BELOW STRICTLY. THE MANUFACTURER IS NOT LIABLE FOR DAMAGES RESULTING FROM IMPROPER USE OR USE INCONSISTENT WITH THIS USER MANUAL. ANY DAMAGES ARISING FROM MISUSE CAN NOT BE THE BASIS FOR COMPLAINTS OR REPAIRS UNDER WARRANTY.

Making any repairs or modifications to the product by an unauthorised person results in loss of warranty and may result in risks to health and life.

#### GENERAL SAFETY RULES

When using electrical devices, take the following safety precautions.

- **NOTE:** To protect against electric shock, do not immerse the power adapter, power cable or breast pump body in water or any other liquid. Protect the device from getting wet, and DO NOT operate the device with wet hands.
- Before you plug in the device for the first time, ensure that the voltage is compatible with your electricity supply by checking the label on the base of the unit. The correct voltage is 100-240 V.
- Connect the device only to an efficient and surge-protected mains supply.
- If needed, use only an adequately connected and protected extension cord.

- Remember:
  1. The extension cord must be adapted to the device's electrical load, according to the information provided on the nameplate.
  2. The extension cord must bear the CE marking.
  3. The extension cord cannot bear signs of any damage.
  4. Arrange the cord so that it does not drape over the countertop or tabletop where it can be pulled by children or tripped over.
    - Do not leave the device unattended when it is plugged into a power socket.
    - When removing the plug from the socket, hold the edge of the socket. Never pull the plug out of the socket by pulling on the power cord.
    - Do not place any power cords under furniture or other obstacles.
    - DO NOT put on or near a hot gas or electric stove, or in a preheated oven. Make sure that the power cords are away from hot surfaces, radiators and central heating pipes. Do not leave the device directly in the sun.
    - Always unplug the device from the socket before cleaning.
    - **NOTE!** Do not use any device with a damaged cord or plug. Do not use the device after its failure or any other damage.
    - Never use an incomplete or broken device. Replace damaged or worn parts immediately. Use only replacement parts provided by the manufacturer.
- Follow the recommendations on the correct use of the device and its accessories.
- This device is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge unless they have been given supervision or instruction concerning the use of the appliance by their guardian.
  - Close supervision is necessary when any appliance is used by a disabled person or near children.
  - ALWAYS keep out of the reach of children.
  - Only use the adapter provided with this product. Do not use other adapters. Do not use the breast pump's power adapter to power other devices.
  - If the power adapter is damaged, replace it with the one purchased from the manufacturer.
  - Do not attempt to repair the device yourself. The device can be repaired by the manufacturer's authorised service centre only.
  - Making any repairs or modifications to the product by an unauthorised person results in loss of warranty and may result in risks to health and life.
  - The manufacturer reserves the right to make design and other modifications to the device.
- NOTE! FAILURE TO FOLLOW THESE INSTRUCTIONS COULD RESULT IN INJURY OR PERMANENT DAMAGE TO YOUR PRODUCT.**
- Use the device only as intended and described in this user manual. Other uses violate the terms of the warranty.
  - Check the product before each use. Never use the product with any signs of damage. In case of damage, contact the manufacturer or distributor service.
  - It is forbidden to use the device if it has been dropped and fallen on a hard surface. In this case, the product shall be returned to the service centre for inspection.
  - Never use the device during the bath.
  - Never use the device in locations exposed to contact with water or other liquids, for example near a bathtub, washbasin, kitchen sink or shower unit.

DO NOT touch the device if it has fallen into water or other liquid. If the device is plugged into a power socket, unplug it immediately.  
**WARNING!** Do not use this product for purposes other than those listed in this user manual. Do not place the product near sources of fire. Before using the breast pump, make sure that all parts were assembled correctly. If you feel breast or nipple discomfort, stop using the breast pump immediately. Wash your hands before using the product. Keep the product out of the reach of newborns and children. Do not drop or hit the product to avoid damage. Do not touch product elements with bare hands immediately after sterilisation in boiling water to avoid burns. Do not resell a used device.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in your area of waste disposal service or the supplier of the product.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures. The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is a Class II appliance.

This product is not a toy. Keep all parts away from children.

**MILK STORAGE**

Milk can be kept in a bottle or special food containers. Freeze only fresh milk. Do not fill a container fully as frozen milk increases its volume. Breast milk can be heated in the same way as any other food (but never in the microwave as this may destroy valuable nutrients and antibodies). Frozen breast milk can be defrosted overnight in the refrigerator and then warmed gradually in a bottle warmer or hot water. After defrosting, shake the container or stir the milk to mix the separated fat particles with the remaining volume of milk. Thawed milk can be used for 24h. It should be stored in a refrigerator. NEVER refreeze previously defrosted milk. DO NOT add fresh breast milk to defrosted milk. DO NOT add fresh breast milk to frozen milk. NEVER heat breast milk by boiling or putting it in the microwave. ALWAYS check the liquid temperature before feeding. Dispose of any milk that has not been used.

	Freshly expressed milk	Thawed milk
Room temperature (+19°C do +25°C)	up to 4h	do not store
Refrigerator (+4°C)	up to 96h	24h (since total thaw)
Freezer (-20°C)	3-6 months	Never refreeze previously defrosted milk.

The breast pump is intended for personal use. It is not refundable or exchangeable. It is covered by the manufacturer's warranty. Remember that the proof of purchase together with the stamped instructions for use are the basis to examine the complaint. EACH BREAST PUMP MODEL IS TESTED IN TERMS OF USABILITY BEFORE IT IS RELEASED FOR SALE. Remember that if you have lactation problems as a result of which your baby is unable to empty the breast, a breast pump may also be ineffective. Keep the instructions of use as it contains important information. The batch code indicated on the packaging.

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### DECLARATION OF CONFORMITY EU No. 3/22

In scope of directives: 2014/35/EU, 2014/30/EU, 2009/125/EC and 2011/65/EU

Manufacturer: Canpol Sp. z o.o.  
Puławska Street 430, 02-884 Warsaw, Poland

We hereby declare under our responsibility that our breast pump with adaptor (cat. no 12/215)

**brand: Canpol babies, breast pump**

is in conformity with the following directives:

- directive 2014/35/EU relating to electrical equipment designed for use within certain voltage limits
- directive 2014/30/EU relating to electromagnetic compatibility
  - directive ErP 2009/125/EC
  - directive RoHS 2011/65/EU

and is in conformity with the following harmonized standards:

EN 60335-1:2012 + A11 :2014 (PN-EN 60335-1:2012+A13 :2017-11+A11 :2014-10)  
EN 62233:2008 (PN-EN 62233:2008)  
EN 55014-1:2006+A1 :2009+A2 :2011 (PN-EN 55014-1:2017-06)  
EN 55014-2:1997+A1 :2001+A2 :2008 (PN-EN 55014-2:2015-06)  
EN 61000-3-2:2014 (PN-EN 61000-3-2:2014-10)  
EN 61000-3-3:2013 (PN-EN 61000-3-3:2013-10)

as well as other technician specifications:

The Commission Regulation (EC) No. 278/2009

The last two digits of the year in which the **CE** marking was affixed: 19

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(name, surname and signature of authorized person, position)

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